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EN

CC DISK PMMA (coloured, 3-layered, Multilayer) is used in CAD/CAM milling machines.

Indications:

- Temporary restorations,
- gingiva formation after placing dental implants,
- accurate determination of the occlusal contacts before placing zirconia prosthetic substitutes.

Contraindications:

- known allergies to PMMA

Intended users: Product is intended to be used by professionals – dental technicians.

Patient selection criteria: Totally or partially edentulous patients.

Technical Information

Flexural strength:	114
Vickers hardness:	26.6 MPa (N/mm ²)
E-modulus:	E 2771 MPa (N/mm ²)

Rules that should be considered:

- The minimal thickness in the occlusal area is 1 mm and in the cervical area 0.6 mm.
- The bond must have at least 10 mm² in the frontal region and at least 15 mm² in the distal region.
- In the distal part is the maximal allowable distance of the two missing teeth.
- When used 3-layered disc pay attention to the positioning the scann of prosthetic appliances regarding to the layers. Enamel layer on the disc is marked

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Polishing and cementation:

- When milling use the burs for acrylics.
- If you are using cooling emulsion during milling, clean prosthetic substitute in an ultrasonic bath.
- Polish only with polishing pastes suitable for acrylic treatment such as Unileesk or Interlesk. Use a soft polishing instrument.
- After polishing clean prosthetic substitute with vapour or in ultrasonic bath.
- Clean the stump and inner part of prosthetic substitute with alcohol or diethyl ether. Cement with temporary cement.
- Ready-Made product can remain in the mouth up to 5 years.

Final safety notes: When processing and polishing particles are produced, therefore it is recommendable to use protective gloves and protective masks with filters FFP1.

Side effects: In very rare cases, a patient may develop an allergy to PMMA. For selecting suitable material the Dentist is responsible.

Notice: Any serious incident, that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Storage: Store at room temperature.

Warranty! Whether given verbally, in writing or by practical instructions, our recommendations for use are based upon our own experience and trials and can only be considered as standard values. Our products are subject to further development. Therefore alterations in construction and composition are reserved.

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