Navodila za uporabo

VMix - Vacuumix

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Instructions for use

VMix - Vacuummix

Device for vacuum mixing of plaster, investment material and silikon.

Interdent d.o.o.

Opekarniška cesta 26, 3000 Celje, Slovenija
Tel: +386/3-425-62-00, Fax: +386/3-425-62-10
E-mail: info@interdent.cc, www.interdent.cc

CONTEN

- WARNING! 3**
- 1 TECHNICAL DATA..... 4**
- 2 DIRECTIONS FOR USE..... 5**
 - 2.1 Switch ON..... 5**
 - 2.2 How to program the device 5**
 - 2.3 Mixing 6**
- 3 IMPORTANT PRACTICAL INFORMATION 8**
 - 3.1 Instructions for lifting and carrying of the device..... 8**
 - 3.2 Conditions of the environment..... 8**
 - 3.3 Cleaning 8**
 - 3.4 Maintenance 8**
 - 3.5 Instructions for assembly..... 8**
- 4 DECLARATION OF CONFORMITY..... 9**
- 5 WARRANTY FOR PRODUCT 0679 9**

WARNING!

BEFORE INSTALLING AND STARTING THE APPLIANCE READ THIS INSTRUCTIONS CAREFULLY! THE APPLIANCE CAN ONLY BE OPERATED BY PERSON WHO IS CAREFULLY INSTRUCTED ABOUT THE OPERATION AND WORKING METHODES.

ATTENTION: The appliance must always be placed on a solid horizontal surface!

Before connecting check, if the rated voltage corresponds to the value of $230 + / - 10\%$ / 50 Hz. The device should be connected to the rated voltage via single-phase outlet with protective earth contact.

CAUTION!

Before handling with device the network must be pulled out of the plug. Any interference into device must be done only by authorized service provider. Otherwise the warranty is no longer valid.

After the repair is done the device is penetrating strength and the protective conductor must be tested. In case of a failure which no longer provides the safe operation of the device, the unintended use must be prevented.

1 TECHNICAL DATA

VMIX Vacuummix is a highly productive vacuum mixer with microprocessor control, designed for mixing of investments, plaster and silicone.

Power supply	220/230 V, 50 Hz
Power consumption	260 rpm
Vacuum pump Capacity	20l/ min
Vacuum pump Controlled vacuum	0,93 Bar
Programmable time for the operating cycle:	
Preliminary vacuuming	0 - 20 s
mixing	0 - 250 s
vacuuming after mixing	0 - 10 s
Weight (without bowls)	10,1 kg
Dimensions (L/B/H)	310 x 210x 330 mm
Bowl size	250, 550, 835 ali 1200 ml

2 DIRECTIONS FOR USE

2.1 Switch ON

Before switching on the device make sure that the voltage is 220 VAC and the grounding of the contact is good.

Power is supplied after switching the main power key which is placed at the bottom of the Mixer. The following greeting message appears on the display for three seconds: »VMix Version 1.3«. (picture 1)

Picture 1

And then you can see the normal indication: »VMix READY for work«. (picture 2)

Picture 2

2.2 How to program the device


In order to enter the programming mode, you have to press the multi-functional knob **one time**. This indication appears on the display: PROGRAMMING, »PrMix Time: XX s«. (picture 3)

Picture 3

This parameter means time for preliminary vacuuming. By rotating the multi-functional knob left/right you will change the value of this parameter respectively upwards and downwards between 0 to 20 seconds.

If the knob is pressed **again**, the display for programming of the next parameter - time for mixing - appears: »Mix Time: XX s«. (picture 4)

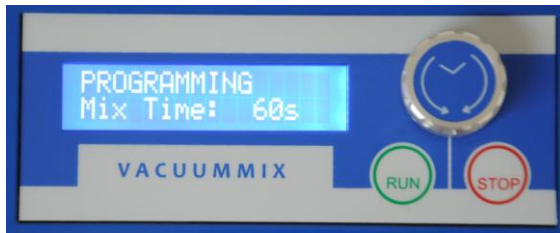Picture 4

The change is from 0 to 250 seconds.

The next pressing of the knob activates a display for programming the parameter - time for vacuuming after mixing: AMix Time: XX s. (picture 5)

Slika 5

The change is from 0 to 10 seconds.

2.3 Mixing

In order to make the real mixing you have to place the investment for mixing in the bowl, put the cover of the bowl, then insert it in the fixator and hold it tight with hand.

Then press the "RUN" button . (picture 6)

Picture 6

The machine starts vacuuming, the build-in buzzer rings and the notice appears on the display for seven seconds: »VMix RUNS GRASPING«. (picture 7)

Picture 7

Hold the bowl tight until this indication disappears and the buzzer stops ringing, after that you can leave hold of it. Now the bowl is held by the vacuum created in it.

Then the actual execution of the program for mixing is to be accomplished as the three phases according the durations of the preliminarily set times are performed. The first and the third phases can be skipped if the programmed times are equal to 0 seconds.


After completion of the program the machine stops. Continuous acoustic signal is produced. The following message is indicated on the display: »END Of Program Push STOP«. (picture 8)

Picture 8

The device is now in mode of waiting the bowl to be taken out.

Hold the bowl tight with hand and press the button "STOP". Now the mixer produces uninterrupted acoustic signal. The note is written on the display: »Hold The Bowl«. (picture 9)

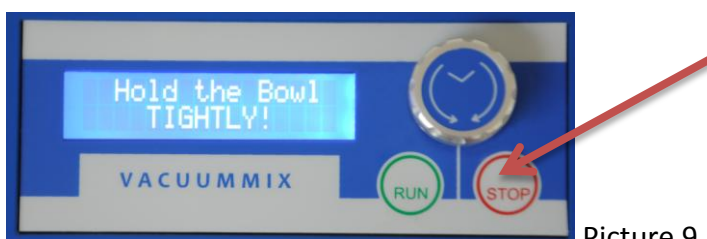Picture 9

In this way the apparatus warns you that air bursts into the bowl. After 7 beep signals the bowl is released from the vacuum and it can be removed from the flange.

In case that during the operation you want to quit the mixing process, you can do it by pressing of the "STOP" button.

NOTE!

You can change the time for mixing many times according to your own needs. The programmed value is saved even after turning off the power, since the device has memory independent from the electricity.

3 IMPORTANT PRACTICAL INFORMATION

3.1 Instructions for lifting and carrying of the device

The mixer has no special handles. When moving is needed, you have take the device with two hands from the bottom side.

3.2 Conditions of the environment

The mixer is designed to work under normal conditions of the environment, i.e.

- 2.1. To be used in buildings to 2000m above sea level.
- 2.2. Surrounding temperature - from 5 to 40 degrees Celsius;
- 2.3. Maximum relative humidity 80% by temperature until 31 degrees Celsius.
- 2.4. Level of contamination - II level.

3.3 Cleaning

Clean only with dry or slightly moisten cloth (no solvents!), only after you have switch off the device from the mains supply.

3.4 Maintenance


- 4.1. Replace the fuses only with ones with the same nominal values, namely:
T3,15A slow (class T).
- 4.2. No maintenance work should be performed by persons who are not authorized by the producer!

3.5 Instructions for assembly

The device is fixed on the wall with special additional plank. The plank is fasten t ight on appropriate height and the mixer is hung up on it through a "bayonet" fitting.

Slika 10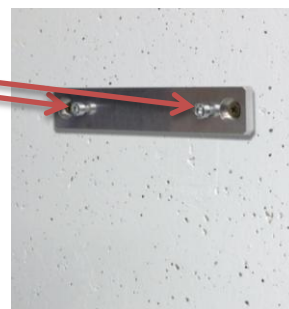

Slika 11

4 DECLARATION OF CONFORMITY

Interdent d.o.o., Opekarniška cesta 26, SI -3000 Celje s polno odgovornostjo izjavljamo, da je izdelek, naveden v tej izjavi skladen z navedenimi EU smernicami

We Interdent d.o.o., Opekarniška cesta 26, SI – 3000 Celje declare that products below are in accordance with EU directives

Naziv izdelka: Vakumski mešalec

Name of the product: Vacuum mixing unit

Tip: VMix - Vacuummix

Type: VMix - Vacuummix

Upoštevane EU smernice:

EU directives:

- EU – nizkonapetostna smernica: 73/23 EEC, 2006/95/EC
EU– low voltage directive: 73/23/ EEC, 2006/95/EC
- EU – smernica za elektromagnetno združljivost: 2004/108/EC
EU – directive for electromagnetic compatibility: 2004/108/EC
- EU – smernice za izgradnjo strojev: 2006/42/EC
EU – directive for machine construction: 2006/42/EC

Testne metode po standardih:

Test methods according standards

- EN61010-1:2010 Requirements for safety of the electrical equipment
- EN61000 Requirements for EMC
- EN55014-1:2007 Requirements for EMC - Emission
- EN55014-2:1999 Requirements for EMC - Immunity

Standart kakovosti: ISO 9001: 2008

Quality standard: ISO 9001:2008

Proizvod je varen za uporabo v standardnih pogojih. Vsaka sprememba proizvoda, ki ni odobrena s strani proizvajalca, razveljavlja to izjavo.

Product is safe in standard conditions. Any change of the product that is not approved by producer, repeals this declaration.

Mesto in datum izdaje: Celje dne 28.11.2013

Place and date of issue: Celje, 28. 11. 2013

Odgovorna oseba:

Responsible person:


Igor Grudnik, inž. elektr.
dipl. oec

Direktor:

Director:

Špela Zagožen Krsnik, univ.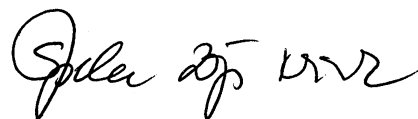

5 WARRANTY FOR PRODUCT 0679

Product:

Serial number.

Date of sale:

Signature of the seller:

Warranty declaration:

- The product will work flawless during the warranty period if being administered in accordance with its purpose and the instructions provided;
- Warranty period expires after 12 months from purchase day. At your request we will repair failure or defects on the product, if you will report it within the warranty period. Failure will be resolved free of charge within 45 days from the date of failure report. The product, which will not be repaired within this deadline, will be, at your request, replaced with a new one. We will extend the warranty period for the time the product was being repaired..
- Validity of the warranty should be proofed by certified warranty card and original invoice.

The warranty is voided:

- If an unauthorized person interferes into the device
- If provided instructions for use are not being considered
- If the device is being mechanically injured
- If the unoriginal spare parts are being build in

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