
Gingi Sil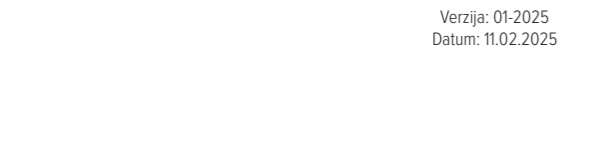

	<div> <div>Interdent d.o.o. - SI - 3000 CELJE</div> <div> <p>Opekarniška cesta 26 · T: +386 (0)3 425-62-00</p> E: info@interdent.cc · www.interdent.cc</div> </div>	
	Made in Slovenia	

Upoštevajte navodila za uporabo / Consult instructions for use / Gebrauchsanweisung beachten / Obratite pozornost na upute za uporabu / Poštovati uputstvo za upotrebu / Sledujte návod k použití / Dodržujte návod na použití / Сназвайте инструкции за употреба / Consulte las instrucciones de uso / Consulter les instructions d'utilisation / Consultare le istruzioni per l'uso / Consultar instruções de utilização / Ααφβύετε υπόψη τις οδηγίες χρήσης / Tartsa be a használati utasítás instrukcióit / Laikytis naudojimo instrukcijas / Beakta bruksanvisningen / Необходимо учитывать инструкции по применению

REF Številka artikla / Catalogue number / Artikelnummer / Kataloški broj / Broj artikla / Katalogové číslo / Číslo výrobku / Артикулен номер / Número de referencia / Référence catalogue / Numero di catalogo / Número de referencia / Αριθμός προϊόντος / Katalógusszám / Artikulo numeris / Artikelnummer / Каталогный номер

LOT Številka serije / Batch code / Chargennummer / Serijski broj / Broj šarže / Kód dávky / Číslo šarže / Номер на партида / Número de lote / Code de lot / Codice del lotto / Número de lote / Αριθμός παρτίδας / Tételszám / Partijos numeris / Batchnummer / Серийный номер

Uporabno do / Use-by-date / verwendbar bis / Može se upotrijebiti do / Použit do / Použite do / Годен до / Upotrebljivo do / Fecha de caducidad / Date limite d'utilisation / Utilizzare entro la data / Válido até / Ημερομηνία λήξης / Lejárati dátum / Tinka naudoti iki / Används före / Использовать до

Proizvajalec / Manufacturer / Hersteller / Proizvodač / Proizvodač / Výrobce / Výrobca / Производител / Fabricante / Fabricant / Fabbricante / Fabricante / Κατασκευαστής / Gyártó / Gyártó / Tillverkare / Производитель

Datum proizvodnje / Date of manufacture / Herstellungsdatum / Datum proizvodnje / Datum proizvodnje / Datum výroby / Datum výroby / Дата на производство / Fecha de fabricación / Date de fabrication / Data di fabbricazione / Data de fabrico / Ημερομηνία κατασκευής / Gyártási datum / Pagaminimo data / Tillverkningsdatum / Дата изготовления

SL:

Gingi Sil

Navodilo za uporabo:

Namen uporabe:

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Instructions for use:

Indications:

Gingi Sil is a poly-addition silicone for the construction of gingival mask. Suitable for use with both the direct method (directly on the impression) and the indirect method (with a silicone template on the working model). After polymerisation, it can be easily milled with drills. It can be scanned without the use of scan spray.

Contraindications:

Do not use Gingi Sil as a conventional impression material in the mouth!

Technical characteristics:

- Colour: pink
- Mixing ratio: 1 : 1
- Working time at 23°C: 1 min
- Setting time at 23°C: 8 min
- Hardness (Shore): 70

Method of use:

Insert the cartridge into the dispenser. When using the cartridge for the first time, squeeze out some material so that both come out evenly. Do not remove the mixing tip from the cartridge after use. Before the next use, replace the old mixing tip with a new one.

- Direct method:

Clean and dry the impression. Isolate the area where you will apply Gingi Sil with a separator () and wait until it is completely dry. Insert the cartridge into the dispenser and attach the mixing tip (for more precise parts, attach the intraoral tip to the mixing tip). To avoid bubbles, do not interrupt dispensing, and leave the tip in the silicone while dispensing. Wait until the silicone has completely hardened, then you can continue making the model.

- Indirect method:

Prepare a silicone template (e.g. Interlabosil) on the uncut plaster working model. The silicone template should extend over the area where the gingival mask will be. Then cut the plaster model and grind the stumps. Drill two holes in the silicone template in the gingival mask area. Apply the separator () to the entire inside of the silicone template and wait until it dries completely. Insert the cartridge into the dispenser and attach the mixing tip. Place the silicone template back on the working model. Dispense Gingi Sil through one of the holes on the silicone template until the silicone flows out of the other hole. Wait until the silicone dries completely, then remove it from the model and template and process it with a sharp knife or drill.

Storage:

Keep the product tightly sealed in a cool dry place.

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