

INDICATIONS

CC DISK PMMA PINK is made out of PMMA with pigments. It is used in CAD/CAM milling machines for making highly esthetic dentures. It can be used for immediate load implant denture or long term provisional denture in combination with acrylic denture teeth.

Denture is designed in CAD Software. 5- axes milling machine is required to reach undercuts.

Acrylic artificial teeth are attached on denture base with PMMA resin or light curing composite material with proper primer.

TECHNICAL INFORMATION

Flexural strength: 114 MPa

Vickers hardness: 26.6

E-modulus: 2771 MPa

Residual monomer: < 1%

WARNING

In very rare cases, a patient may develop an allergy to PMMA . For selecting suitable material the Dentist is responsible.

STORAGE: Store at room temperature.

Diameter of the discs: 94-100 mm

Thickness of the discs: 25 - 30 mm;

Colour: pink

Warranty!

Whether given verbally, in writing or by practical instructions, our recommendations for use are based upon our own experience and trials and can only be considered as standard values. Our products are subject to further development. Therefore alterations in construction and composition are reserved.

Manufacturer:



OPEKARNIŠKA CESTA 026, SI – 3000 CELJE

tel: +386/34256200; fax: +386/34256210

e-mail: info@interdent.cc; www.interdent.cc