navodila za uporabo

Interdrill System

Interdent d.o.o.

Opekarniška cesta 26, 3000 Celje, Slovenija
Tel: +386/3-425-62-00, Fax: +386/3-425-62-10
E-mail: info@interdent.cc, www.interdent.cc

Kazalo vsebine

1 UPORABA	4
2 TEHNIČNI PODATKI.....	4
3 OPREMA.....	4
4 MONTAŽA SVEDRA.....	4
5 PRILAGAJANJE GLOBINE VRTANJA.....	5
6 VRTANJE	5
7 ČIŠČENJE POSODICE ZA ZBIRANJE PRAHU	5
8 KONSTRUKCIJA APARATA	6
9 VZDRŽEVANJE.....	7
10 IZJAVA O SKLADNOSTI	8
11 GARANCIJSKI LIST	9

OPOZORILO!

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11 WARRANTY FOR PRODUCT REF.: 0680..... 9

CAUTION!

BEFORE INSTALLING AND STARTING THE APPLIANCE READ THIS INTRUCTIONS CAREFULLY! THE APPLIANCE CAN ONLY BE OPERATED BY PERSON WHO IS CAREFULLY INSTRUCTED ABOUT THE OPERATION AND WORKING METHODES.

ATTENTION: The appliance must always be placed on a solid horizontal surface!

Before connecting check, if the rated voltage corresponds to the value of $230 + / - 10\% / 50 \text{ Hz}$. The device should be connected to the rated voltage via single-phase outlet with protective earth contact.

WARNING!

Before handling with device the network must be pulled out of the plug. Any interference into device must be done only by authorized service provider. Otherwise the warranty is no longer valid.

After the repair is done the device is penetrating strength and the protective conductor must be tested. In case of a failure which no longer provides the safe operation of the device, the unintended use must be prevented.

1 USE

Compact and ergonomic device for precise mechanical positioning and drill revolution regulation. User has the possibility of precise mechanical positioning of the working model and regulating the drill revolutions. In the event of use for other purposes, manufacturer does not guarantee for the safe operation and for the damage on the appliance, materials and people.

Interdrill system has an automatic switching on system of the motor during the pressure on working table which gives the possibility for the device to be in working position for a long period of time without disturbing your work. The laser system provides the indication of drilling area and enables precise and excellent visibility of the laser marker on the model even in light working space. Optimal drilling speed can be achieved by regulating the drill revolutions. Foldable support of the device makes it handy for carriage and storage and assures comfortable work even in a sitting position. Dusting system maintains your working environment cleaner and more comfortable. Dust container is integrated in the working table and it is easy to remove and clean.

2 TECHNICAL DATA

- Electrical supply.....230 V \pm 10% / 50 Hz
- Max power100 W
- Max drill revolutions4000 min⁻¹
- Collet diameter \varnothing 3 mm
- Drill regulating movement.....12 mm
- Laser class.....3
- Dimensions (height, width, length).....160x410x157 mm
- Weight.....5 kg

3 EQUIPMENT

- Interdrill device **REF 0680**
- Pin drill 2.0mm **REF 0318**
- Key 2 kom
- Fuse 2 kom

Supplied with pin drill suitable for twin pins REF 0317 and REF 0319.

4 DRILL INSTALLATION

1. Before installing the drill, make sure the main switch is OFF (position 0).
2. Unscrew the chuck with the keys size 17 and 19, insert the drill through the table hole into the chuck so the tip of the drill is below the table.

5 ADJUSTING THE DRILLING DEPTH

1. Drill the hole into the non working model which you will not need.
2. With a help of chosen pin check the depth of the drilled hole.
3. If necessary enhance or reduce the hole depth by screwing the regulating screw at the back of the device. To reduce the drilling depth rotate the screw clockwise/right.
4. Repeat the procedure 1. and 2.

6 DRILLING

WARNING!

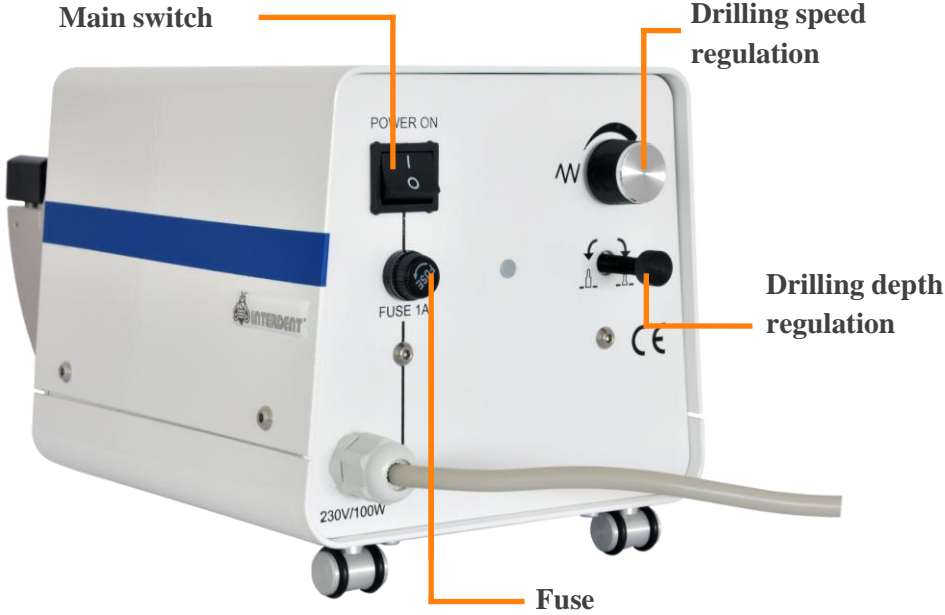
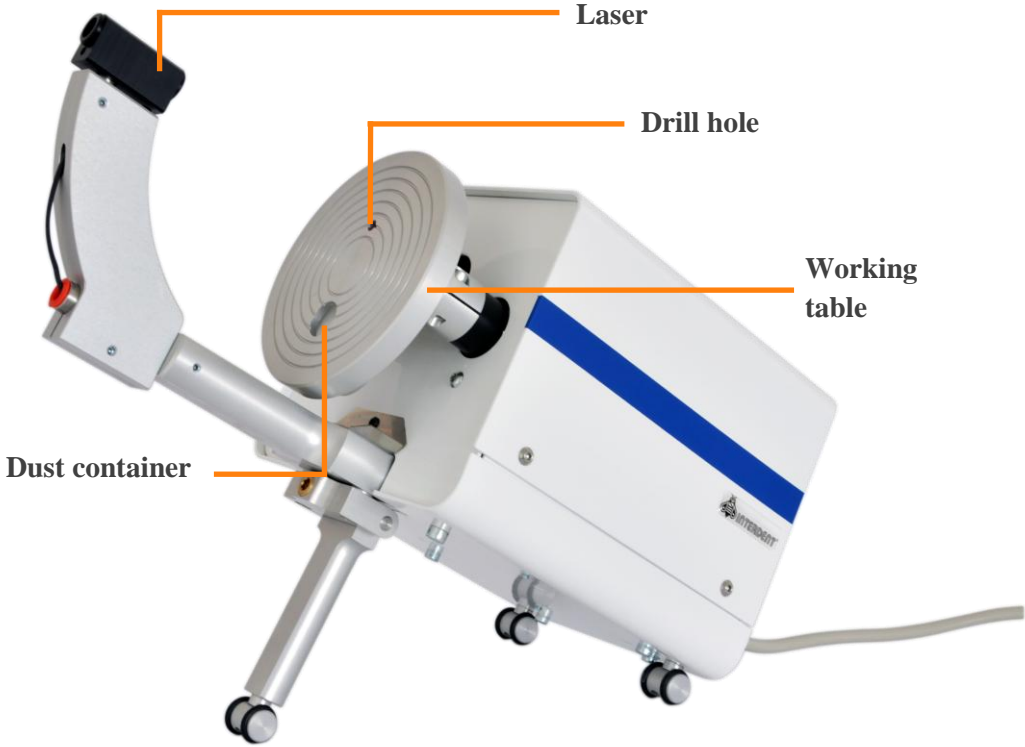
Before starting the working model has to be properly prepared. Model should be sanded perpendicular to the axis of the pins. The surface has to be completely flat.

1. Put the plaster model on the working table and hold it from all sides.
2. With the help of laser carefully position the drilling site.
3. Press the model by the table. The device automatically turns ON when pressing the table and automatically turns OFF when the table returns to the starting position.

7 CLEANING THE DUST CONTAINER

1. Before installing the drill make sure the main switch is OFF (position 0).
2. At the bottom of the table in the dust container area turn the hold into horizontal position and remove the dust container. Cleaned container insert back to it`s position and turn the hold back to vertical position.

8 DEVICE CONSTRUCTION

9 MAINTANACE

WARNING!

Before performing any maintenance on the device disconnect the power supply!

Always take care the device is clean. Housing of the device can be wiped by the dry or slightly moisture cloth. Special attention should be paid on cleaning the drilling area. We recommend annual review by an authorized service.

10 DECLARATION OF CONFORMITY

Interdent d.o.o., Opekarniška cesta 26, SI -3000 Celje s polno odgovornostjo izjavljamo, da je izdelek, naveden v tej izjavi skladen z navedenimi EU smernicami

We Interdent d.o.o., Opekarniška cesta 26, SI – 3000 Celje declare that products below are in accordance with EU directives

Naziv izdelka: Naprava za vrtanje modelov
Name of the product: Drilling unit

Tip: Interdrill
Type: Interdrill

Upoštevane EU smernice:
EU directives:

- EU – nizkonapetostna smernica: 73/23 EEC, 2006/95/EC
EU– low voltage directive: 73/23/ EEC, 2006/95/EC
- EU – smernica za elektromagnetno združljivost: 2004/108/EC
EU – directive for electromagnetic compatibility: 2004/108/EC
- EU – smernice za izgradnjo strojev: 2006/42/EC
EU – directive for machine construction: 2006/42/EC

Testne metode po standardih:

Test methods according standards

- EN61000-3-2 ed. 3: 2006 +A1:2010 + A2: 2010
- EN61000-3-3 ed. 2:2009
- EN61010-1:2003, EN61010-2-10 ed. 2:2004
- EN55011 ed. 2: 2007, EN 55011 ed. 3: 2010
- EN ISO 12100-1,2 A1. 2004

Standart kakovosti: ISO 9001: 2008
Quality standard: ISO 9001:2008

Proizvod je varen za uporabo v standardnih pogojih. Vsaka sprememba proizvoda, ki ni odobrena s strani proizvajalca, razveljavlja to izjavo.

Product is safe in standard conditions. Any change of the product that is not approved by producer, repeals this declaration.

Mesto in datum izdaje: Celje dne 21.11.2012

Place and date of issue: Celje, 21. 11. 2012

Odgovorna oseba:
Responsible person:

Igor Grudnik, inž. elektr.
dipl. oec

Direktor:

Director:

Špela Zagožen Krsnik, univ.

11 WARRANTY FOR PRODUCT REF.: 0680

Product:.....

Serial number.

Date of sale:

Signature of the seller:

Warranty declaration:

- The product will work flawless during the warranty period if being administered in accordance with its purpose and the instructions provided;
- Warranty period expires after 12 months from purchase day. At your request we will repair failure or defects on the product, if you will report it within the warranty period. Failure will be resolved free of charge within 45 days from the date of failure report. The product, which will not be repaired within this deadline, will be, at your request, replaced with a new one. We will extend the warranty period for the time the product was being repaired..
- Validity of the warranty should be proofed by certified warranty card and original invoice.

The warranty is voided:

- If an unauthorized person interferes into the device
- If provided instructions for use are not being considered
- If the device is being mechanically injured
- If the unoriginal spare parts are being build in

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