Potrebno upoštevati navodila za uporabo / Consult instructions for use / Bitte beachten Sie die Bedienungsanleitung / Viz návod k použití /
Potrebno poštivati uputstva za upotrebu / Consultar las instrucciones de uso

Uporabno do / Use by / Gültig bis / Použítí / Upotrijebiti do / Usar antes

LOT

Kontrolna številka / Batch number / Seriennummer / Seriové číslo /
Serjski broj / Numero de serie

REF

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GB

CC DISK PMMA transparent is used in CAD/CAM milling machines for making PMMA constructions for casting metal frameworks and for modelation for press ceramic works. Milled construction from CC Disk PMMA transparent can be tested in patient mouth.

Indications:

- Milled modelation for metal construction casting
- Milled modelation for pressed ceramic.
- Testing of milled modelation in patient mouth before casting.
- Testing of milled modelation before making zirconium framework

Product is 100% organic and burns out without residues.

Final safety notes:

When processing and polishing particles are produced, therefore it is recommendable to use protective gloves and protective masks with filters FFP1.

Side effects:

In very rare cases, a patient may develop an allergy to PMMA. For selecting suitable material the Dentist is responsible.

Storage:

Store at room temperature.

Warranty!

Whether given verbally, in writing or by practical instructions, our recommendations for use are based upon our own experience and trials and can only be considered as standard values. Our products are subject to further development. Therefore alterations in construction and composition are reserved.

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