CC DISK PMMA PINK is made out of PMMA with pigments. It is used in CAD/CAM milling machines for making highly esthetic dentures.

It can be used for immediate load implant denture or long term provisional denture in combination with acrylic denture teeth. Denture is designed in CAD Software. 5- axes milling machine is required to reach undercuts. Acrylic artificial teeth are attached on denture base with PMMA resin or light curing composite material with proper primer.

**Indications:**

- for production of base for total prostheses
- for immediate load implant denture or long term provisional denture in combination with acrylic denture teeth.

**Intended users:** Product is intended to be used by professionals – dental technicians.

**Patient selection criteria:** Totally or partially edentulous patients.

**TECHNICAL INFORMATION**

Flexural strength: 114 MPa

Vickers hardness: 26.6

E-modulus: 2771 MPa

Residual monomer: < 1%

Diameter of the discs: 94-100 mm

Thickness of the discs: 25 - 30 mm;

Colour: pink

**Final safety notes:** When processing and polishing particles are produced, therefore it is recommendable to use protective gloves and protective masks with filters FFP1.

**Warning:** In very rare cases, a patient may develop an allergy to PMMA . For selecting suitable material the Dentist is responsible.

**Side effects:** In very rare cases, a patient may develop an allergy to PMMA. For selecting suitable material the Dentist is responsible.

**Notice:** Any serious incident, that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

**Storage:** Store at room temperature.

**Warranty!**

Whether given verbally, in writing or by practical instructions, our recommendations for use are based upon our own experience and trials and can only be considered as standard values. Our products are subject to further development. Therefore alterations in construction and composition are reserved.

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